

# Freezpoint™

## DEVICE ACTIVATION



### STEP 1



Remove Freezpoint™ from sealed pouch.



### STEP 2




While grasping the device with both hands, tighten the activation knob **CLOCKWISE no more than one rotation**. The blue dot on the activation knob should align with the raised diamond on the body of the device.



### STEP 3



Freezpoint™ is now activated and ready to use.

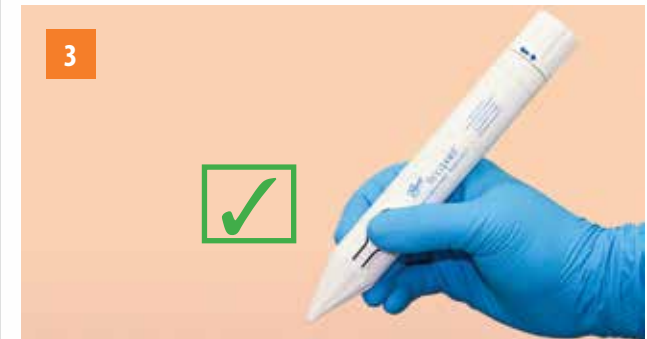

Do not rotate the activation knob in a counter-clockwise direction once Freezpoint™ is activated.

# Freezpoint™

## ADMINISTER TREATMENT "ACTUATION"



### STEP 1



Position the patient such that the surface to be treated is exposed and facing upwards.



#### **PRE-TREATMENT HELPFUL HINT:**

Be sure the area to be treated is well hydrated. Dry skin is not optimal for Cryosurgery. For example, soak hands in water or moisten skin with a damp cloth.



### STEP 2



While holding Freezpoint™ in an upright position, hold the spray nozzle over the lesion at 1cm distance. Press the actuation button with the tip of the thumb to administer treatment.



### STEP 3



Spray cryogen in a small circular motion over the lesion. White frost (as shown below) should be visible on patient's skin for the entire duration of treatment.



#### **HELPFUL HINT:**

- Cryosurgery outcomes may improve when target lesions are treated by pulsing two on/off cycles.
- Minimize freezing the margin around the lesions treated for cosmetic reasons.